

Quick dispenser pocket reference for

NUPOLAR[®]

polarized lenses

Benefits:

- *Blocks “blinding glare” for increased comfort and safety*
- *Better vision and safety, less eye fatigue*
- *Protection from sun and UVA and UVB*
- *Excellent choice for light-sensitive eyes*
- *Vibrant colors and improved contrast*
- *Stylish look*

Perceived Substitutions:

- *Tinted Lenses: do not block blinding glare*
- *Clip-ons: increase weight of eyewear, may scratch lenses and decrease optical quality*
- *Fit-ons: frames worn over existing glasses are heavy and uncomfortable, may limit and distort vision, and wearers look unstylish*

Dispensing Tips:

- *Discuss the above benefits*
- *Use the NuPolar Glare Demonstrator to show how polarizers work to block glare*
- *Send patient home with the NuPolar “Look” Card to experience in their own environment*

**Ask for
NUPOLAR[®]
By Name!**



Quick dispenser pocket reference for

NUPOLAR®

polarized lenses



Features:

- *Quality Optics*
- *High efficiency polarizer*
- *1.8 mm minimum center thickness*
- *Tintable gray 1, neutral grey, and rich brown colors*
- *Lightweight plastic: 1.498 R.I.*
- *High impact polycarbonate: 1.586 R.I.*
- *Conforms to ANSI Z80.1 and ANSI Z80.3*
- *100% UV blockage (ANSI Z80.3)*
- *Transmission = 15% - 18% visible*
- *Image® progressive, flat top and single vision lens available*
- *Accommodate a wide range of prescriptions*
- *Optical Laboratories Association Award:
BEST IN LENS TECHNOLOGY*

Processing Tips:

- *Finish and tint as normal*
- *Match horizontal indicators to 180° within 3.0° for each lens*
- *No acetone or alcohol applied to edges*
- *Glaze lenses snug in frame, not tight*
- *A.R., mirror coat compatible*
- *Backside coatings approved*
- *Groove with caution, away from the polarizing film*