

STRIPPING HARD COATING FROM Younger Optics Clear Trilogy[®] Lenses

The following instructions pertain to clear product ONLY.

Equipment Set Up:

Tank 1


10% solution of
Sodium Hydroxide
in DI water by weight,
at 55° C (131° F)
with ultrasonics

Tank 2

DI Water

Tank 3

DI Water

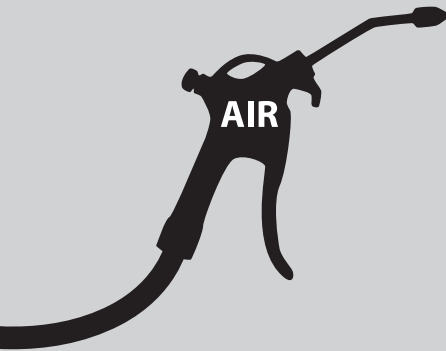

Process:

1. Mount lens in a stainless steel holder and immerse it in **Tank 1** for 8 minutes.
2. Rinse the lens in **Tank 2** for 1 minute with agitation.
3. Rinse the lens in **Tank 3** for 1 minute with agitation.
4. Air dry the lens using compressed air.
5. Clean the lens with isopropyl alcohol and a soft tissue or cloth.