

Seamless™ 28


Fitting Guide



The Younger Seamless lens was the first commercially available lens of this kind. It was the first bifocal to offer the cosmetic benefits of being invisible, while attempting to correct some of the common dysfunctions of a segmented bifocal, such as jump and split images.

It is also the easiest invisible product to fit and dispense. There are no monocular measurement requirements and processing is as simple and easy as with conventional bifocals. Seamless 28 is designed to accommodate the cosmetic needs of your patients for that no-line look.

- The Seamless 28 lens should be fit 1mm above the lower lid. If you routinely fit to the top of the lower lid, take your measurement to this point and add 1mm to that height.
- Correct PD measurement is important.
- When checking the Rx power and axis set the aperture of your lensmeter 8mm above the transition zone (checking at the transition zone will result in unwanted cylinder power).
- When checking for add power set the aperture in the center of the reading area.

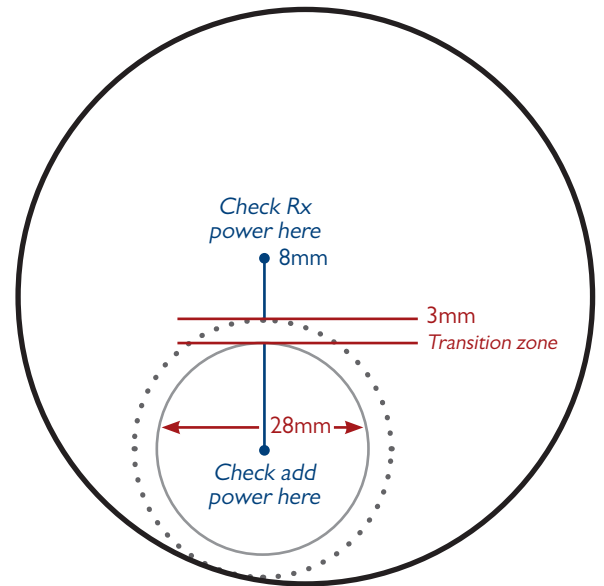

YOUNGER OPTICS
The Optical Lens Innovators

Seamless is a trademark of Younger Mfg. Co.

SEAMLESS™ 28 CLEAR HARD RESIN AVAILABILITY

DIAMETER	NOMINAL CURVE	TRUE CURVE	BACK CURVE	EDGE	CENTER	SEG INSET	SEG DROP	SAG AT 50	RADIUS	ADD RANGE	RX RANGE
76	2.25	2.24	4.25	16.0	13.5	0	4	1.32	236.61	0.75-3.00	-8.00D to +6.00D
76	4.25	4.23	6.25	12.5	9.5	0	4	2.52	125.30	0.75-3.00	
76	6.25	6.31	6.25	8.0	8.0	0	4	3.80	83.99	0.75-3.00	
76	8.25	8.35	6.25	8.0	11.5	0	4	5.13	63.47	0.75-3.00	